


# bolink™ Large Cap Gravity Feeding Set with ENFit™ Connector



**Indications for Use:** The U Deliver Bolink ENFit Enteral Feeding Sets are intended for over-the-counter use to deliver liquid nutritional formulas or water to a patient's enteral access device (feeding tube).

**Device Description:** The Bolink™ large cap gravity feeding set consists of tubing (A) with a clamp (B), a screw cap (C), an ENFit™ connector (D) for connecting to feeding tubes and a protective cap (E).

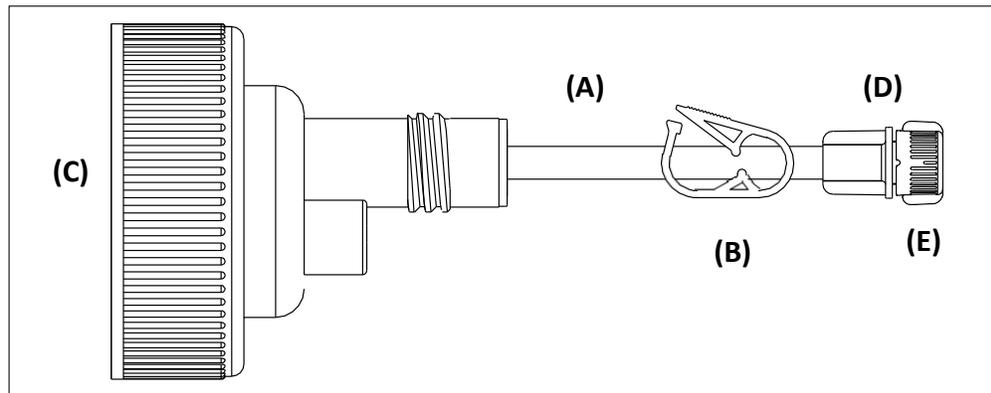

The Bolink™ large cap gravity feeding set is intended to connect compatible, commercially available collapsible food containers with 40 mm diameter connectors to feeding tubes with ENFit™ proximal connectors.

Not made with natural rubber latex. Not formulated with DEHP or BPA.

## Warnings:

- For enteral feeding only.
- This device incorporates an ENFit™ Connector which is designed to reduce the likelihood of tubing misconnections; however, the potential to misconnect this device with connectors of other healthcare applications still exists.
- Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision.
- Do not use with non-collapsible food containers.

## Precautions:

- This device should be discarded after 24 hours and replaced with a new device.
- For single patient use only.
- Dispose with household waste. Do not recycle.


**Instructions for Use:**

1. Confirm feeding tube position and flush following manufacturer’s instructions.
2. Close clamp (B) on Bolink™ Tubing (A).
3. Remove cap on collapsible food container, attach screw cap (C) to filled food container and confirm cap is secured tightly.
4. Open clamp (B) and squeeze collapsible food containers to fill Bolink tubing (A).
5. When tubing is filled, close clamp (B).

**WARNING:** Before connecting to a feeding tube, confirm that the Bolink tubing is being connected to a feeding tube only. DO NOT connect to non-feeding tube. This includes confirming that any transitional connector with a Christmas Tree Tip used are connected to a feeding tube and not IV tubing.

6. Remove protective cover (E), use ENFit® connector (D) to attach Bolink tubing to feeding tube. If the feeding tube has a funnel instead of an ENFit connector use a transition connector with Christmas tree tip.
7. Open clamp (B), hold food container above feeding tube to start gravity flow, and, if necessary, squeeze container to control delivery rate.
8. When finished, close clamp (B), disconnect Bolink tubing from feeding tube and collapsible food container.
9. Flush Bolink tubing (A) with tap water by holding under a faucet and letting water flow through the tubing until the water runs clear.
10. 9. Flush feeding tube following manufacturer’s directions.

Connect this end to feeding tube.

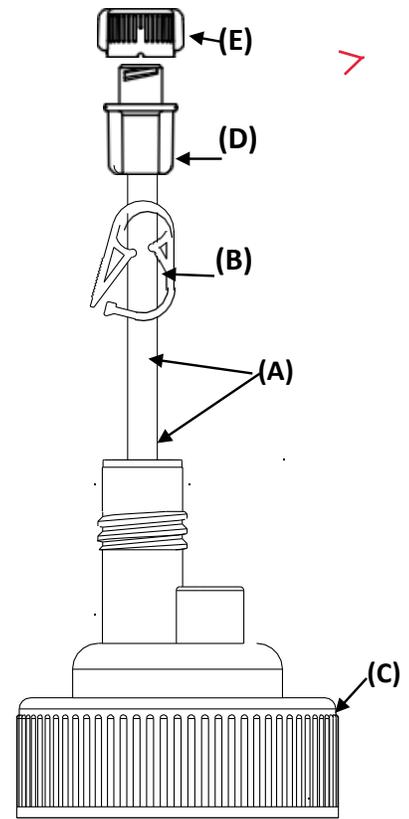

Connect this end to collapsible food container.

**Symbol Glossary:**

Catalogue number. Indicates the manufacturer's catalogue number so that the medical device can be identified. Ref. 5.1.6 in ISO 15223-1:2012		Use-by date. Indicates a medical device that has to be used by its expiration date. Ref. 5.1.4 in ISO 15223-1:2012	
Non sterile. Indicates a medical device that has not been subjected to a sterilization process. Ref. 5.2.7 in ISO 15223-1:2012		Consult instructions for use. Indicates the need for the user to consult the instruction for use. Ref. 5.4.3 in ISO 15223-1:2012	
Keep Dry. Indicates a medical device that needs to be protected from moisture. Ref. 5.3.4 in ISO 15223-1:2012		Do not use if package is damaged. Indicates a medical device that should not be used if the package has been damaged or opened. Ref. 5.2.8 in ISO 15223-1:2012	
Do not reuse. Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure. Ref. 5.4.2 in ISO 15223-1:2012		Manufacturer. Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC. Ref. 5.1.1 in ISO 15223-1:2012	
Batch code. Indicates the manufacturer's batch code so that the batch or lot can be identified. Ref. 5.1.5 in ISO 15223-1:2012			

**Note:** Where referenced, symbols are specified in ISO 15223-1:2012 Medical devices — Symbols to be used with medical device labels, labelling and information to be supplied — Part 1: General requirements.

For more information on Bolink™ tubing and U Deliver Medical products, visit [www.UDeliverMedical.com](http://www.UDeliverMedical.com).

 U Deliver Medical, LLC.  
7900 Excelsior Blvd, Suite 108  
Hopkins, MN 55343  
Made in Italy

Bolink™ is a trademark of U Deliver Medical. ENFit™ is a trademark of GEDSA, used with permission. ©2019 U Deliver Medical, LLC